ENGLISH

- Material: Aluminium
- Weight Limit: 100kg
- Length: 250 / 400mm

	Torque	Loctite
Set Screws	15 Nm	243

For secure fastening use the loctite recommended and apply rated torque twice.


CUTTING AND ASSEMBLY OF ALUMINUM TUBE:

The tube should be shortened according to measurements. Avoid damage to tube as it may prevent easy insertion into Tube Clamp Adapter.

- Clamp tube with a 30mm cutting jig.
- Ensure tube is cut straight with a hacksaw or tube cutter.
- Remove all burr with a file before inserting into Tube Clamp Adapter.
- Ensure tube is fully inserted and in firm contact with base of Tube Clamp Adapter.

- 2 year limited warranty.

Rated to 100kg / 220lbs, is not recommended for active use.

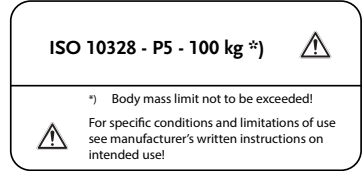A weatherproof device allows use in a wet and/or humid environment but does not allow submersion. Fresh water splashing against the enclosure from any direction shall have no harmful effect. Dry thoroughly after contact with fresh water. Fresh water: Includes tap water. Excludes salt and chlorinated water.

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EN – Caution: Össur products and components are designed and tested according to the applicable official standards or an in-house defined standard when no official standard applies. Compatibility and compliance with these standard is achieved only when Össur products are used with other recommended Össur components. If un-usual movement or product wear is detected in a structural part of a device at any time, the patient should be instructed to immediately discontinue use of the device and consult his/her clinical specialist. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients. If any problems occur with the use of this product, immediately contact your medical professional.

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