ENGLISH

- Material: Titanium/Carbon
- Weight Limit: 166kg
- Length: 250 / 400mm

	Torque	Loctite
Set Screws	15 Nm	243

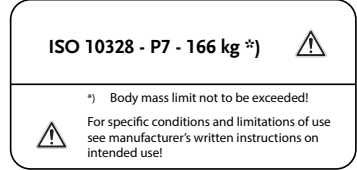
For secure fastening use the loctite recommended and apply rated torque twice.

Cutting and assembly of Carbon Tube:
The tube should be shortened according to measurements. Avoid damage to tube as it may prevent easy insertion into Tube Clamp Adapter.

- Remove Tube Insert.
- Clamp tube with a 30mm cutting jig.
- Ensure tube is cut straight with a fine tooth saw.
- Remove all burs with a file.
- Replace Tube Insert before inserting into Tube Clamp Adapter.
- Ensure tube is fully inserted and in firm contact with base of Tube Clamp Adapter.

2 year limited warranty.

Rated to 166kg / 366lbs, is not recommended for active use.

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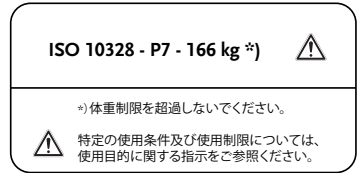
EN – Caution: Össur products and components are designed and tested according to the applicable official standards or an in-house defined standard when no official standard applies. Compatibility and compliance with these standard is achieved only when Össur products are used with other recommended Össur components. If un-usual movement or product wear is detected in a structural part of a device at any time, the patient should be instructed to immediately discontinue use of the device and consult his/her clinical specialist. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients. If any problems occur with the use of this product, immediately contact your medical professional.

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www.ossur.com ©Copyright Össur 2019 IFU 0094 1402_001 Rev. 7