ENGLISH

A-762121 & A-762111 (Aluminium), A-766122, A-766112, A-766121, A-766111 (Carbon / Titanium)

Part number: **A-762121 , A-762111**

Contents
Female Pylon
Female Pyramid Tube Clamp

- Material: Aluminium
- Weight limit: 100kg
- Length: 400 mm / 250 mm

Part number: **A-766122, A-766112, A-766121, A-766111**

Contents
Female Pylon
Female Pyramid Tube Clamp
Carbon Tube Insert
Male Pyramid Socket Adapter (A-766122, A-766112)

- Material: Carbon / Titanium
- Weight limit: 166 kg
- Length: 400 mm / 250 mm

	Torque	Locite
Pinch Bolt	10 Nm x 2	242
Set Screws	15 Nm	242

NOTE: Pinch bolt must be positioned anteriorly on the prosthesis. Ensure tube is fully inserted into tube clamp adapter before securing.

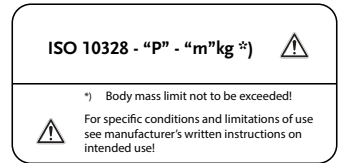
Procedure for cutting and assembling pylon:

The pylon should be shortened based on the Individuals measurements. This must be done according to the instructions below, avoid damaging the pylon as this may prevent easy insertion into the tube clamp adapter.

- Do not clamp tube directly in vice, use either 30 mm-tube clamp or tube cutter.
- Ensure tube is cut straight with a hacksaw, not at an angle.
- Remove all burs with a file and add Carbon Tube Insert before inserting into tube clamp adapter.
- Loosen pinch bolt on tube clamp adapter before inserting pylon.
- Ensure tube is fully inserted and in firm contact with base of tube clamp adapter.

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EN – Caution: Össur products and components are designed and tested according to the applicable official standards or an in-house defined standard when no official standard applies. Compatibility and compliance with these standard is achieved only when Össur products are used with other recommended Össur components. If un-usual movement or product wear is detected in a structural part of a device at any time, the patient should be instructed to immediately discontinue use of the device and consult his/her clinical specialist. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients. If any problems occur with the use of this product, immediately contact your medical professional.

Össur Americas 27051 Towne Centre Drive Foothill Ranch, CA 92610, USA Tel: +1 (949) 382 3883 Tel: +1 800 233 6263 ossurusa@ossur.com	Össur Canada 2150 – 6900 Graybar Road Richmond, BC V6W 0A5 , Canada Tel: +1 604 241 8152	Össur APAC 2F, W16 B No. 1801 Hongmei Road 200233, Shanghai, China Tel: +86 21 6127 1707 asia@ossur.com
Össur UK Ltd Unit No 1, S.Park Hamilton Road Stockport SK1 2AE, UK Tel: +44 (0) 8450 065 065 ossuruk@ossur.com	Össur Nordic Box 770 191 27 Sollentuna, Sweden Tel: +46 1818 2200 info@ossur.com	Össur Australia 26 Ross Street, North Parramatta NSW 2151 Australia Tel: +61 2 88382800 infosydney@ossur.com
Össur Europe BV De Schakel 70 5651 GH Eindhoven The Netherlands Tel: +800 3539 3668 Tel: +31 499 462840 info-europe@ossur.com	Össur Iberia S.L.U Calle Caléndula, 93 - Miniparc III Edificio E, Despacho M18 28109 El Soto de la Moraleja, Alcobendas Madrid – España Tel: 00 800 3539 3668 orders.spain@ossur.com orders.portugal@ossur.com	Össur South Africa Unit 4 & 5 3 on London Brackenage Business Park Brackenfell 7560 Cape Town South Africa Tel: +27 0860 888 123 infosa@ossur.com
Össur Deutschland GmbH Augustinusstrasse 11A 50226 Frechen Deutschland Tel: +49 (0) 2234 6039 102 info-deutschland@ossur.com	Össur Europe BV – Italy Via Baroaldi, 29 40054 Budrio, Italy Tel: +39 05169 20852 orders.italy@ossur.com	

Instructions for Use

FEMALE PYLON KIT