- Material: Titanium
- Weight Limit: 166kg
- Weight: 58g

1. Isolate cast with PVA bag.
2. Apply Double layer of Perlon Stockinet.
3. Apply single 50mm carbon tapes crossed over distal end of cast, in AP/ML direction.
4. Position Socket Adapter and Lamination Cap on model in correct alignment and rotation.
5. Apply Lay up material and combinations of material appropriate to the individuals weight and impact/activity level.
6. Layers of material may be trimmed to different lengths to provide a thinner lighter proximal socket.
7. Lamination material should be tied into groove on Socket Adapter.
8. Laminate with 80:20 acrylic lamination resin.

- 2 year limited warranty.

ISO 10328 - P7 - 166 kg *) 

*) Body mass limit not to be exceeded!

 For specific conditions and limitations of use see manufacturer's written instructions on intended use!

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EN – Caution: Össur products and components are designed and tested according to the applicable official standards or an in-house defined standard when no official standard applies. Compatibility and compliance with these standard is achieved only when Össur products are used with other recommended Össur components. If un-usual movement or product wear is detected in a structural part of a device at any time, the patient should be instructed to immediately discontinue use of the device and consult his/her clinical specialist. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients. If any problems occur with the use of this product, immediately contact your medical professional.

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www.ossur.com ©Copyright Össur 2019 IFU 0097 1165_001 Rev. 8