ENGLISH

- Material: Steel
- Weight limit: 166kg
- Weight: 124g

	Torque	Loctite
Pinch Bolt	10 Nm	243

For secure fastening use the loctite recommended and apply rated torque twice.

INSTRUCTIONS FOR INTEGRATION INTO LAMINATED SOCKET

Note: For more complex positions, the socket can be laminated in two layers. The pronged adapter should be shaped and fixed to the inner layer with putty. All surfaces should be roughened prior to the second lamination.


When shaping the pronged adapter, care must be taken to prevent disortion of the distal threads.

- Isolate cast with PVA bag
- Apply Double layer of Perlon Stockinet
- Apply single 50mm carbon tapes crossed over distal end of cast, in AP/ML direction
- Shape Prong Adapter and position on distal end of model in correct alignment and rotation
- Screw Lamination cap into distal threads
- Apply clay to Pinch Bolt and Pinch Bolt housing to protect from resin
- Tie 50mm carbon tapes into grooves on Adapter, position in AP/ML postions
- Lay up material and combinations of material should be appropriate to the individuals
- Layers of material may be trimmed to different lengths to provide a thinner lighter proximal socket
- Tie 50mm carbon tapes into grooves on Adapter, position in AP/ML positions
- Apply double layer of Perlon Stockinet
- Laminate with 80:20 acrylic lamination resin

The Lamination Cap should be removed following lamination by unscrewing from Prong Adapter.

- 2 year limited warranty**

Rated to 166 kg/366 lbs, for normal use.

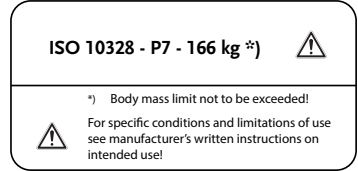A weatherproof device allows use in a wet and/or humid environment but does not allow submersion. Fresh water splashing against the enclosure from any direction shall have no harmful effect. Dry thoroughly after contact with fresh water. Fresh water: Includes tap water. Excludes salt and chlorinated water.

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EN – Caution: Össur products and components are designed and tested according to the applicable official standards or an in-house defined standard when no official standard applies. Compatibility and compliance with these standard is achieved only when Össur products are used with other recommended Össur components. If un-usual movement or product wear is detected in a structural part of a device at any time, the patient should be instructed to immediately discontinue use of the device and consult his/her clinical specialist. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients. If any problems occur with the use of this product, immediately contact your medical professional.

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www.ossur.com ©Copyright Össur 2019 IFU 1132_001 Rev. 6