ENGLISH

- Material: Steel
- Weight limit: 45kg
- Weight: 79g

	Torque	Loctite
Pinch Bolt	5 Nm	243

For secure fastening use the loctite recommended and apply rated torque twice.

Instructions for integration into laminated socket.

Note: For more complex positions, the socket can be laminated in two layers. The pronged adapter should be shaped and fixed to the inner layer with putty. All surfaces should be roughened prior to the second lamination.


When shaping the pronged adapter, care must be taken to prevent disortion of the distal threads.

- Isolate cast with PVA bag
- Apply Double layer of Perlon Stockinet
- Apply single 50mm carbon tapes crossed over distal end of cast, in AP/ML direction
- Shape Prong Adapter and position on distal end of model in correct alignment and rotation
- Screw Lamination cap into distal threads
- Apply clay to Pinch Bolt and Pinch Bolt housing to protect from resin
- Tie 50mm carbon tapes into grooves on Adapter, position in AP/ML postions
- Lay up material and combinations of material should be appropriate to the individuals
- Layers of material may be trimmed to different lengths to provide a thinner lighter proximal socket
- Tie 50mm carbon tapes into grooves on Adapter, position in AP/ML positions
- Apply double layer of Perlon Stockinet
- Laminate with 80:20 acrylic lamination resin

The Lamination Cap should be removed following lamination by unscrewing from Prong Adapter.

2 year limited warranty

Rated to 45 kg/100 lbs, for normal use.

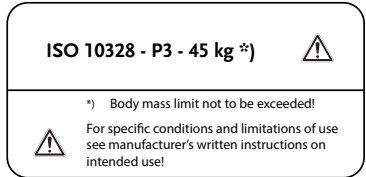A weatherproof device allows use in a wet and/or humid environment but does not allow submersion. Fresh water splashing against the enclosure from any direction shall have no harmful effect. Dry thoroughly after contact with fresh

water. Fresh water: Includes tap water. Excludes salt and chlorinated water.

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www.ossur.com ©Copyright Össur 2019 IFU 0193 IFU 1205_001 Rev. 6
Össur Deutschland GmbH