ENGLISH

- Material: Aluminium.
- Weight Limit: 100kg.
- Weight: 75g.

	Torque	Loctite
Mounting Screws	10 Nm	243

For secure fastening use the loctite recommended and apply rated torque twice.
May be used for Transtibial and Transfemoral laminated sockets.

INSTRUCTIONS FOR INTEGRATION INTO LAMINATED SOCKETS.

Socket must be laminated in two layers. The 4-Hole Socket Adapter should be attached to the inner layer with putty. All surfaces should be roughened prior to second lamination.

1. Isolate cast with PVA bag.
2. Apply appropriate lay up material for first lamination.
3. Apply outer PVA bag and laminate.
4. Once cured, remove PVA bag and roughen outer surface of lamination.
5. Attach 4-Hole socket adapter in correct alignment and rotation with putty.
6. Position Lamination Cap on Adapter and secure using a 4mm Hex key.
7. Seal putty and Lamination Cap with sealing resin.
8. Apply Lay up material and combinations of material appropriate to the individuals weight and impact/ activity level.

IMPORTANT: Tie 50mm carbon reinforcements into both grooves, between lamination cap and adapter, and groove in adapter

The Lamination Cap should be removed following lamination by unscrewing the bolts using a 4mm Hex key.

- **2 year limited warranty.**

Rated to 100kg / 220lbs, is not recommended for active use.

ISO 10328 - P5 - 100 kg *)

*) Body mass limit not to be exceeded!

 For specific conditions and limitations of use see manufacturer's written instructions on intended use!

A weatherproof device allows use in a wet and/or humid environment but does not allow submersion. Fresh water splashing against the enclosure from any direction shall have no harmful effect. Dry thoroughly after contact with fresh water. Fresh water: Includes tap water. Excludes salt and chlorinated water.

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