Instructions for Use

CONNECTION PLATE

EN		Instructions for Use	3
DE		Gebrauchsanweisung	5
FR		Notice d'utilisation	7
ES		Instrucciones para el uso	9
IT		Istruzioni per l'uso	11
NL		Gebruiksaanwijzing	13
PT		Instruções de Utilização	15
TR		Kullanım Talimatları	17

DESCRIPTION OF THE PRODUCT

The Össur connection plate is a part of leg prostheses and serves to connect the socket with the locking mechanism for silicone sockets.

Important: When choosing the adapter please observe the approved maximum body weight. Add possible extra loads to the patient's body weight (e.g. bottle crates) in order not to exceed the maximum body weight.

TECHNICAL DATA

Art. No.	Own weight	Weight limit	Material	Nm
1.651.014	61 g	136 kg	Al	12

THE FOLLOWING POINTS MUST BE OBSERVED WHEN USING THE ÖSSUR CONNECTION PLATE:

1. The connection plate must be degreased with solvent before gluing.
2. The connection plate must lie flat on the base of the socket.
3. The adapter mounted distally on the connection plate must lie flat.
4. The securing screws must be tightened to 12Nm and secured with Loctite.
5. CAUTION: deformation of any kind leads to premature wear-and-tear (fracture) of the connection plate.

MAINTENANCE

Warning

The Össur Adapter must be inspected at least every 6 months. Inspect for:

- The general condition of the adapter
- The alignment
- The suitability of the patient (e.g. max. body weight, degree of mobility)
- Damage to the adapter, external condition
- If the user falls the complete prosthesis system must be meticulously inspected
- The torque of the connecting screws if applicable

GENERAL CONDITIONS:


This product is recommended for use by one patient only. Product liability pursuant to the Medical Devices Act becomes null and void if it is used for treating more than one patient.

- The compatibility of Össur products with fittings from other manufacturers is guaranteed as long as the following points are satisfied:
- The fittings are only used in compliance with their intended purpose (see Instructions for Use).
- If fittings with different max. weight limits are used, the maximum loading corresponds to that of the weakest fitting.
- The use of tested individual components does not relieve the prosthetist from his duty to check by all possible means the combination of fittings for their suitability, correct assembly and safety.
- If there are any indications that a certain combination of fittings does not guarantee the required level of safety, then the fittings must not be combined.
- The prostheses must be built in accordance with the generally recognised professional rules of the orthopaedic technical trade.
- If the patient's situation changes (e.g. body weight, degree of activity, etc.) the prosthesis must be checked immediately.
- If the prosthesis has been subjected to disproportionately high stresses, e.g. during a fall, it must be inspected for any possible damage immediately by a specialist workshop.
- Safety-relevant regulations for individual fittings (e.g. maintenance intervals, see Instructions for Use) must be observed.

COMBINATIONS OF DIFFERENT FITTINGS:

The above general conditions apply, in addition:

- Use only fittings that comply with the requirements of ISO 10328.
- Use only fittings that comply with the requirements of the Medical Devices Act.
- The fittings must be designed for use with standard pyramid or 30 mm systems, and must be compatible with each other.

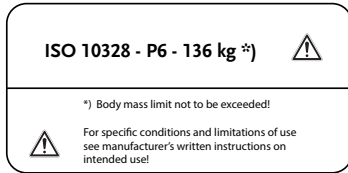GUARANTEE

For Össur connection plate, there is a guarantee period of 24 months.

SAFETY WARNINGS

- You must observe the alignment guidelines and instructions for use.
- You must observe the maintenance and care guidelines.
- This medical aid is for use by one patient only.

USEFUL LIFE

The maximum useful life of the Össur adapter is 36 months.

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EN – Caution: Ossur products and components are designed and tested according to the applicable official standards or an in-house defined standard when no official standard applies. Compatibility and compliance with these standards is achieved only when Ossur products are used with other recommended Ossur components. If an unusual movement or product wear is detected in a structural part of a device at any time, the patient should be instructed to immediately discontinue use of the device and consult his/her clinical specialist. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients. If any problems occur with the use of this product, immediately contact your medical professional.

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