ENGLISH

JUNIOR 4-HOLE MALE PYRAMID

AJ-233100

- Material: Aluminum/Stainless Steel
- Weight Limit: 45kg
- Weight: 39g

	Torque	Loctite
Mounting Screws	6 Nm	243

For secure fastening use the loctite recommended and apply rated torque twice.

- **2 year limited warranty**

Rated to 45 kg/100 lbs, for normal use.

ISO 10328 - P3 - 45 kg *)

*) Body mass limit not to be exceeded!

For specific conditions and limitations of use see manufacturer's written instructions on intended use!

A weatherproof device allows use in a wet and/or humid environment but does not allow submersion. Fresh water splashing against the enclosure from any direction shall have no harmful effect. Dry thoroughly after contact with fresh water. Fresh water: Includes tap water. Excludes salt and chlorinated water.

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EN – Cautión: Össur products and components are designed and tested according to the applicable official standards or an in-house defined standard when no official standard applies. Compatibility and compliance with these standard is achieved only when Össur products are used with other recommended Össur components. If un-usual movement or product wear is detected in a structural part of a device at any time, the patient should be instructed to immediately discontinue use of the device and consult his/her clinical specialist. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients. If any problems occur with the use of this product, immediately contact your medical professional.

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©Copyright Össur 2019