ENGLISH

PRODUCT DESCRIPTION

The height adjustable pylon is a device to be used as a component within a prosthetic system, establishing a connection between the adapter on a prosthetic socket and the adapter on a prosthetic foot.

DEVICE SPECIFICATIONS

Height Adjustable Pylon - Junior AJ-542350

Material: Aluminium

Height: 76 mm

Height range: 76 – 102 mm

Weight: 145 g

Weight limit: 45 kg

SAFETY INSTRUCTIONS

Warnings:

- This product has been designed and tested based on single patient usage.
- Do not use this device for multiple patients.
- The device should be used with Össur adapters. The durability of the device cannot be guaranteed when non-Össur adapters are used.
- If unusual movement or product wear is detected in a structural part of a device at any time, the patient should be instructed to immediately discontinue use of the device and consult his/her clinical specialist.
- Device is not waterproof. Do not submerge in water.
- If any problems occur with the use of this product, a medical professional needs to be consulted.

SYSTEM ASSEMBLY

Torque and Fastening:

- Torque M4 pinch bolts to 5 Nm.
- Torque M6 set screws to 4 Nm.

For secure fastening:

- Apply Loctite 243 to screws and bolts.
- Apply rated torque twice.

INSTRUCTIONS FOR EXTENSION/RETRACTION

- Loosen the tube clamp pinch bolt for the inner pylon to spin freely.
- Rotate the inner pylon to expand/retract pylon.
- After adjusting the desired length and appropriate toe-in/out, retighten the tube clamp pinch bolt.

Set screw torque values also apply to set screws for extreme alignment.

Position pinch bolts anteriorly on the prosthesis.

ENVIRONMENTAL CONDITIONS

Weatherproof:

A weatherproof device allows use in a wet and/or humid environment but does not allow submersion. Fresh water splashing against the enclosure from any direction shall have no harmful effect. Dry thoroughly after contact with fresh water. Fresh water: Includes tap water. Excludes salt and chlorinated water.

Rinse the product with fresh water and then dry thoroughly after use in a wet and/ or humid environment.

COMPLIANCE

Össur products and components are designed and tested according to the applicable official standards or an in-house defined standard when no official standard applies. Compatibility and compliance with these standards are only achieved if Össur products are used with other recommended Össur components.

This component has been tested according to ISO 10328 standard to 3 million load cycles. Depending on the amputee's activity this corresponds to a duration of use of 3 to 5 years. We recommend carrying out regular yearly safety checks.

ISO 10328 - P3 - 45 kg *)

*) Body mass limit not to be exceeded!
For specific conditions and limitations of use see manufacturer's written instructions on intended use!

DISPOSAL

All components of the product and packaging should be disposed of in accordance with respective national environmental regulations. Users should contact their local governmental office for information on how these items can be disposed of in an environmentally sound manner.

LIABILITY

The manufacturer recommends using the device only under the conditions specified and for the intended purposes. The device must be maintained according to the instructions for use.

The manufacturer is not liable for damage caused by component combinations that were not authorized by the manufacturer.

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EN – Caution: Össur products and components are designed and tested according to the applicable official standards or an in-house defined standard when no official standard applies. Compatibility and compliance with these standard is achieved only when Össur products are used with other recommended Össur components. If un-usual movement or product wear is detected in a structural part of a device at any time, the patient should be instructed to immediately discontinue use of the device and consult his/her clinical specialist. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients. If any problems occur with the use of this product, immediately contact your medical professional.

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©Copyright Össur 2018