




Instructions for Use

BALANCE™ FOOT J

JBPExyz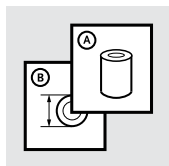


		3
EN	Instructions for Use	5
DE	Gebrauchsanweisung	8
FR	Notice d'utilisation	11
ES	Instrucciones para el uso	14
IT	Istruzioni per l'uso	17
NO	Bruksanvisning	20
DA	Brugsanvisning	23
SV	Bruksanvisning	26
EL	Οδηγίες χρήσης	29
FI	Käyttöohjeet	32
NL	Gebruiksaanwijzing	35
PT	Instruções de Utilização	38
PL	Instrukcja użytkowania	41
JA	取扱説明書	44
ZH	中文说明书	47
KO	사용 설명서	50

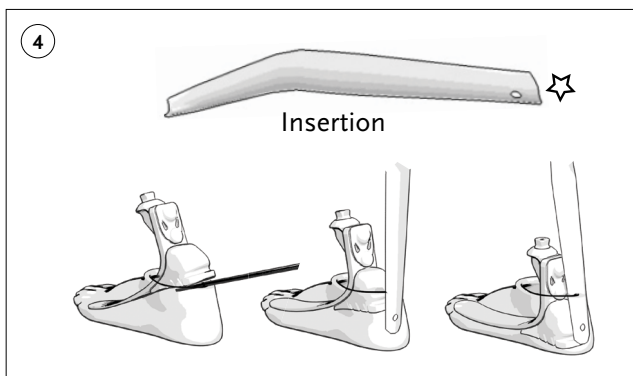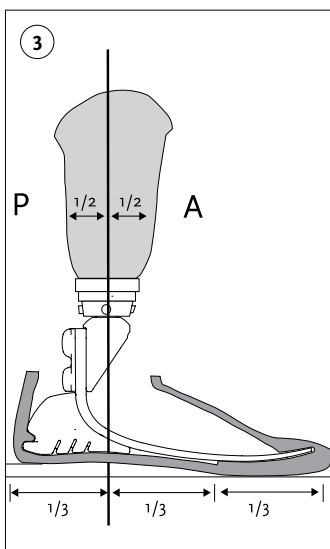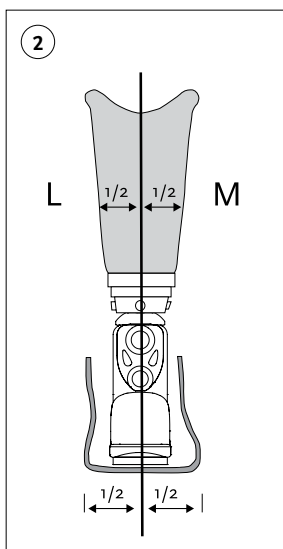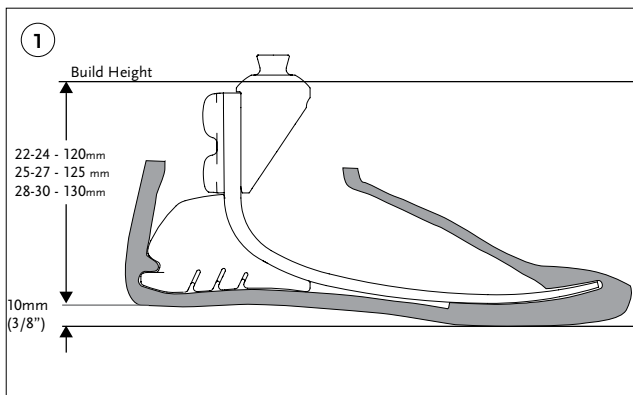5

Removal

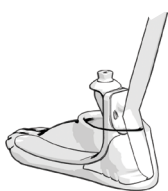ENGLISH

Balance Foot J is referred to as **the device** in the following document. This document provides important information on the indications for use, and handling of the device. It is intended for use by a certified prosthetist and the user of the device.

TECHNICAL SPECIFICATIONS (Figure 1)

- Male pyramid connection
- Clearance: 120mm (22-24)
125mm (25-27)
130mm (28-30)
- Net weight: Size 27, cat2 w/ pyramid: 322g
- Foot cover size 27: 254g
- Maximum user weight: 136kg (300lbs).
- Available in sizes 22-30cm.

INDICATION

Lower limb amputation

CONTRAINDICATION

None Known.

INTENDED USE

The device is a non-invasive, reusable prosthetic device, designed for single patient use. The device is to be used exclusively for the exo-prosthetic fitting of amputations of the lower limb. The device supports low impact levels.

ALIGNMENT: Figure 2

BENCH ALIGNMENT (Figure 2 & 3)

1. Fit the foot cover (**Figure 4 & 5**).
2. Adjust to appropriate heel height (using the shoe).
3. Introduce appropriate socket angles.
4. The load line should fall through the 1/3rd mark on the brim of the foot cover. The mark represents the junction of the posterior and the middle third portion of the foot.

Dynamic Alignment

The foam heel provides cushioning at heel strike. The toe provides forward progression at toe off.

The heel to toe action can be influenced by:

- A-P positioning of foot
- Dorsi-Plantarflexion

CATEGORY SELECTION CHART

Please refer to the selection charts below to determine the appropriate stiffness required according to Össur recommendations.

Weight kg	45-61	62-80	81-97	98-114	115-136
Weight lbs	99-135	136-176	177-214	215-251	252-300
Size 22	1	2	N/A	N/A	N/A
Size 23	1	2	3	4	N/A
Size 24	1	2	3	4	N/A
Size 25	1	2	3	4	5
Size 26	1	2	3	4	5
Size 27	1	2	3	4	5
Size 28	N/A	2	3	4	5
Size 29	N/A	N/A	3	4	5
Size 30	N/A	N/A	N/A	4	5

SHOEHORN

When removing and installing the cover use shoehorn to avoid damage to foot or cover (Figure 4 & 5).

WEATHERPROOF

A weatherproof device allows use in a wet and/or humid environment but does not allow submersion. Fresh water splashing against the enclosure from any direction shall have no harmful effect. Dry thoroughly after contact with fresh water. Fresh water: Includes tap water. Excludes salt and chlorinated water.

LIABILITY

The manufacturer recommends using the device only under the specified conditions and for the intended purposes. The device must be maintained according to the instructions for use. The manufacturer is not liable for damage caused by component combinations that were not authorized by the manufacturer.

COMPLIANCE

This device has been tested according to ISO 10328 standard to two million load cycles. Depending on the user’s activity this corresponds to a duration of use of two to three years. We recommend carrying out yearly safety checks. In the standard mentioned, test levels (P) are assigned to a certain maximal body masses (m in kg). In some cases, which are marked with *, no test level is assigned to the product related maximal body mass. In these cases, the test loads have been adapted adequately on the basis of the specified load level.

ISO 10328 - “P” - “m”kg *)

For specific conditions and limitations of use see manufacturer’s written instructions on intended use!

ISO 10328 - label			
Category	Weight (Kg)	Weight (lbs)	Label text
0	44	97	ISO 10328-P3-44-kg
1	52	115	ISO 10328-P3-52-kg
2	59	130	ISO 10328-P3-59-kg
3	68	150	ISO 10328-P3-68-kg
4	77	170	ISO 10328-P4-77-kg
5	88	194	ISO 10328-P4-88-kg

WARRANTY

Össur provides a warranty of 24 months for the device, and 6 months for the foot cover.

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EN – Caution: Össur products and components are designed and tested according to the applicable official standards or an in-house defined standard when no official standard applies. Compatibility and compliance with these standard is achieved only when Össur products are used with other recommended Össur components. If un-usual movement or product wear is detected in a structural part of a device at any time, the patient should be instructed to immediately discontinue use of the device and consult his/her clinical specialist. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients. If any problems occur with the use of this product, immediately contact your medical professional.

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