

Instructions for Use

CHEETAH® JUNIOR &
CHEETAH® XPLORE JUNIOR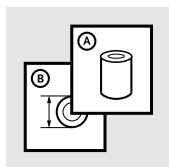


		3
EN	INSTRUCTIONS FOR USE	4
DE	GEBRAUCHSANWEISUNG	7
FR	NOTICE D'UTILISATION	10
ES	INSTRUCCIONES PARA EL USO	13
IT	ISTRUZIONI PER L'USO	16
DA	BRUGSANVISNING	19
SV	BRUKSANVISNING	22
NL	GEBRUIKSAANWIJZING	25
PT	INSTRUÇÕES DE UTILIZAÇÃO	28
PL	INSTRUKCJA UŻYTKOWANIA	31
JA	取扱説明書	34

1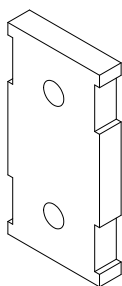


2


Cheetah Xplore Junior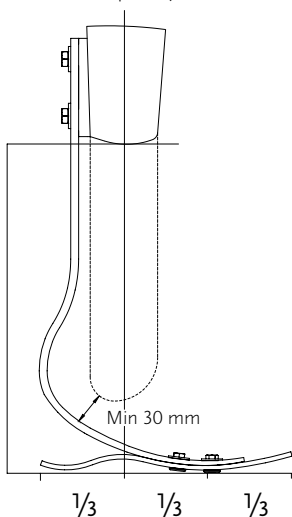

Cheetah Junior

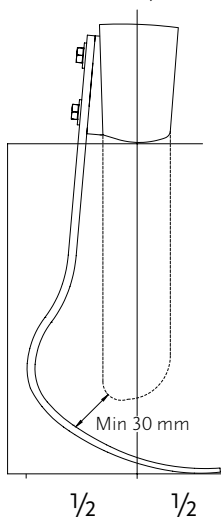3

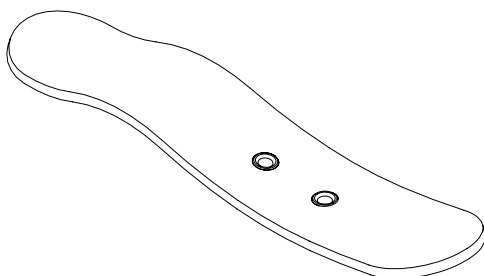ENGLISH

CHEETAH® XPLORE JUNIOR & CHEETAH® JUNIOR

Cheetah System

- Custom foot model for sport and recreational activities.
- Maximum build height
 - Cheetah Xplore Junior 311 mm (12 1/4")
 - Cheetah Junior 307 mm (12 1/16")
- Minimum build height
 - Cheetah Xplore Junior 185 mm (7 5/16")
 - Cheetah Junior 182 mm (7 3/16")
- Maximum weight 55 kg (121 lbs)

CHEETAH XPLORE

Sole plate attachment:

- Grind the sole plate to appropriate size, using the grinding template included in the foot kit (**Figure 3**). The size of the foot is 2 cm shorter when used with the footcover. This needs to be taken into account when grinding the sole plate.
- Use bolts (included in foot kit) to attach sole plate to foot module. Apply Loctite 410 to all threads and torque to following specifications: 14 Nm
- Follow alignment of Cheetah Junior below (**Figur 2**).

Note: If foot will be used without shoes, proper sole material needs to be bonded to the sole plate of the Cheetah Xplore Junior for protection of the foot module.

LAMINATION CONNECTOR (Figure 1)

CHX50001

- Lamination connector is laminated directly into the posterior socket wall.
- Placement of the lamination connector is recommended in the middle 1/3 of the socket for easier height adjustment.
- Make sure the socket surface is flat over the lamination connector to ensure good load distribution on the foot module when connected.

Bolts go through blade into lamination connector in socket. Apply Loctite 410 to all threads and torque to following specifications:

Category	Bolt Size	Torque
1-4	5/16"	28 Nm (21 ft-lbs)

The Cheetah Xplore Junior and Cheetah Junior may also be connected by direct lamination.

FRICTION PADS

Always use friction pads between foot module and socket.

BENCH ALIGNMENT

Sagittal plane (Figure 2)

Cheetah Junior & Cheetah Xplore Junior:

- Determine vertical socket position in alignment jig.

Cheetah Junior:

- Load line bisects socket and falls through the ground contact point.

- Clamp foot into a 5–7° flexion position.

Cheetah Xplore Junior:

- Adjust to appropriate heel height.
- Introduce appropriate socket angles flexion/ extension and abduction/adduction.
- The load line should fall at the junction of the posterior and the middle third portions as shown in **(Figure 2)**.

Coronal plane

- Introduce appropriate socket abduction/adduction using a jig.
- Determine appropriate internal or external socket rotation.

Note: The recommendations for bench alignment are used as a starting point.

CATEGORY SELECTION CHART

Please refer to the selection charts below to determine the appropriate stiffness required according to Össur recommendations.

Cheetah Junior

WEIGHT KG	15-24	25-34	35-45	46-55
WEIGHT LBS	33-53	54-75	76-99	100-121
High to Extreme Impact Level	1	2	3	4

Cheetah Xplore Junior

WEIGHT KG	15-24	25-34	35-45	46-55
WEIGHT LBS	35-53	54-75	76-99	100-121
Moderate to Extreme Impact Level	1	2	3	4

CUTTING FOOT MODULE

- Ensure kids have enough clearance during knee flexion.
- Trim foot module to the top end of the laminating connector.
- Cut foot module using a hack saw or band saw.

WATERPROOF

A waterproof device allows use in a wet and/or humid environment and temporary (<30 mins.) submersion of up to 1m in fresh water. Exposure to fresh water under these circumstances shall have no harmful effect. Dry thoroughly after contact with fresh water. Fresh water: Includes tap water. Excludes salt and chlorinated water.

The components used with the foot need to be waterproof as well.

SOLE ATTACHMENT

- For tighter fit of the sole or to reduce noise bond insert on lower surface of foot module using Loctite 495, clean bonding surfaces before applying adhesive, keep under pressure for 10 minutes cure.
- Slide the Nike sole on to the foot and make sure all the clips go over the foot.
- Pull the rubber strap of the sole over the medallion and slide it under. When detaching the sole, start by pulling the strap away from the medallion, then slide the sole off the foot.

NOTE: For more information see the IFU for Nike Sole.

LIABILITY

The manufacturer recommends using the device only under the specified conditions and for the intended purposes. The device must be maintained according to the instructions for use. The manufacturer is not liable for damage caused by component combinations that were not authorized by the manufacturer.

COMPLIANCE

This component has been tested according to ISO 10328 standard to two million load cycles. Depending on the amputee’s activity this corresponds to a duration of use of two to three years. We recommend carrying out regular yearly safety checks

ISO 10328 - “P” - “m”kg *)

*) Body mass limit not to be exceeded!

For specific conditions and limitations of use see manufacturer’s written instructions on intended use!

In the standard mentioned, test levels (P) are assigned to a certain maximal body masses (m in kg). In some cases, which are marked with, no test level is assigned to the product related maximal body mass.

In these cases, the test loads have been adapted adequately on the basis of the specified load level.

ISO10328					
Category	Weight (Kg)				
1	24	ISO 10328 -	P3	24	kg
2	34	ISO 10328 -	P3	34	kg
3	45	ISO 10328 -	P3	45	kg
4	55	ISO 10328 -	P3	55	kg

- This manual is intended for use by a certified prosthetist.

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EN – Caution: Össur products and components are designed and tested according to the applicable official standards or an in-house defined standard when no official standard applies. Compatibility and compliance with these standard is achieved only when Össur products are used with other recommended Össur components. If un-usual movement or product wear is detected in a structural part of a device at any time, the patient should be instructed to immediately discontinue use of the device and consult his/her clinical specialist. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients. If any problems occur with the use of this product, immediately contact your medical professional.

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