




Instructions for Use

---

FLEX-FOOT<sup>®</sup> ASSURE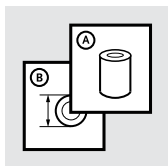

|    |                             |    |
|----|-----------------------------|----|
|    |                             | 3  |
| EN | INSTRUCTIONS FOR USE        | 4  |
| DE | GEBRAUCHSANWEISUNG          | 8  |
| FR | NOTICE D'UTILISATION        | 12 |
| ES | INSTRUCCIONES PARA EL USO   | 16 |
| IT | ISTRUZIONI PER L'USO        | 20 |
| NO | BRUKSANVISNING              | 24 |
| DA | BRUGSANVISNING              | 28 |
| SV | BRUKSANVISNING              | 32 |
| EL | ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ              | 36 |
| FI | KÄYTTÖOHJEET                | 40 |
| NL | GEBRUIKSAANWIJZING          | 44 |
| PT | INSTRUÇÕES DE UTILIZAÇÃO    | 48 |
| PL | INSTRUKCJA UŻYTKOWANIA      | 52 |
| CS | NÁVOD K POUŽITÍ             | 56 |
| TR | KULLANIM TALIMATLARI        | 60 |
| RU | ИНСТРУКЦИЯ ПО ИСПОЛЬЗОВАНИЮ | 64 |
| JA | 取扱説明書                       | 68 |
| ZH | 中文说明书                       | 72 |
| KO | 사용 설명서                      | 76 |
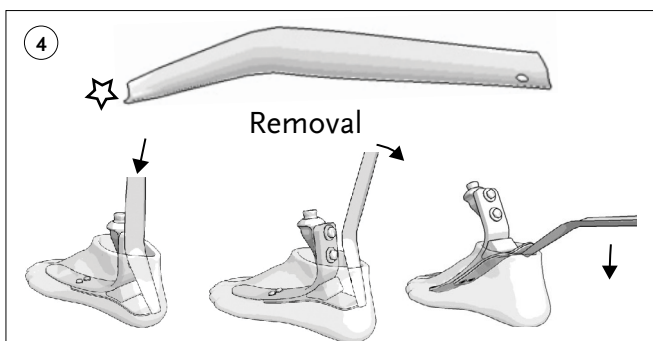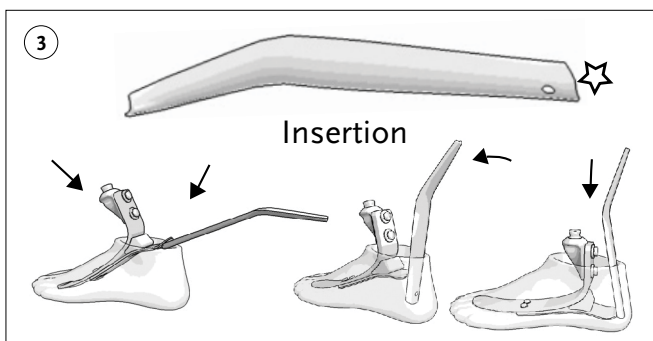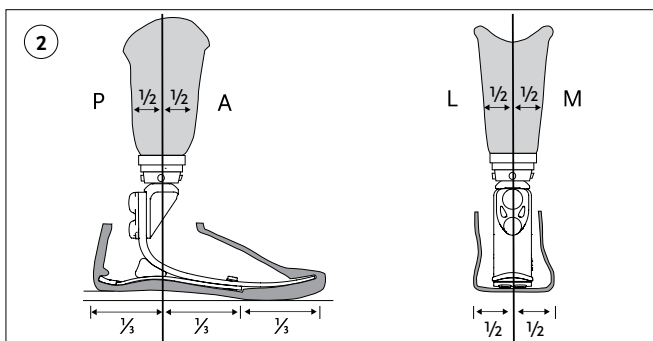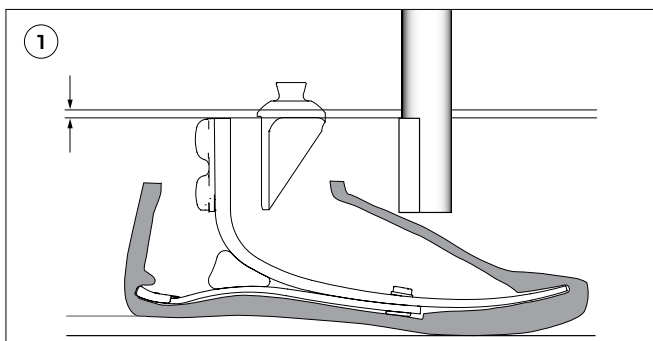
# ENGLISH

---

**Flex-Foot® Assure is to be used exclusively for lower limb amputees.**

## **TECHNICAL SPECIFICATIONS: (Figure 1)**

### ***Tube option***

- Sizes 19–24 MIN clearance 124 mm (4 7/8")
- Sizes 25–27 MIN clearance 119 mm (4 11/16")
- Sizes 28–30 MIN clearance 124 mm (4 7/8")

### ***Pyramid option***

- Sizes 19–24 MIN clearance 120 mm (4 4/3")
- Sizes 25–27 MIN clearance 125 mm (4 15/16")
- Sizes 28–30 MIN clearance 130 mm (5 1/8")

### ***Net weight***

- Size 27, cat2 w/ pyramid: 332g (11.7oz)
- Foot cover size 27: 243g (8.6oz)

### ***Weight limit:***

- MIN 45kg (99lbs)
- MAX 136kg (300lbs)

### ***Available in sizes 19–30cm***

## **INDICATIONS FOR USE**

- Flex-Foot Assure is rated for low impact levels (community ambulators).
- Not suitable for sport and high-impact activities.

## **ALIGNMENT: (Figure 2)**

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- Provides smooth roll over and improved cushioning at heel strike.
- The heel wedge is an essential component for Flex-Foot Assure function.

**! THE HEEL WEDGE SHOULD ALWAYS BE USED WITH THE FOOT.**

## **BENCH ALIGNMENT**

1. Fit the foot cover **(Figure 3 & 4)**.
2. Adjust to appropriate heel height (using the shoe).
3. Introduce appropriate socket angles.
4. The load line should fall through the 1/3<sup>rd</sup> mark on the brim of the foot cover. The mark represents the junction of the posterior and the middle third portion of the foot.

## **DYNAMIC ALIGNMENT**

- The heel stores energy at heel strike and slowly releases it to assist tibial progression.
- The forward momentum results in the toe being loaded for optimum energy release at toe off.
- The heel wedge greatly influences Flex-Foot Assure dynamic functions.

### **Heel too hard**

#### ***Symptoms***

- Rapid heel to toe movement.

- Poor control of the prosthesis at heel strike.
- Minimal energy return feeling.
- Knee may buckle.

#### *Actions*

- Shift socket posteriorly or foot anteriorly.
- Plantar-flex the foot.
- Review category selection.

### **Heel too soft**

#### *Symptoms*

- Foot will come to flat position too early (sinking into a hole feeling).
- Extra energy is required to climb up over the toe.
- Toe will feel too stiff.
- Knee may hyperextend.

#### *Actions*

- Shift socket anteriorly or foot posteriorly.
- Dorsi-flex the foot.
- Review category selection.

### **PROTECTIVE FILM ON PYRAMID**

The protective film protects the pyramid from scratching. Remove the film when the fitting is complete.

### **CATEGORY SELECTION CHART**

Please refer to the selection charts below to determine the appropriate stiffness required according to Össur recommendations.

| WEIGHT KG  | 45-61  | 62-80   | 81-97   | 98-114  | 115-136 |
|------------|--------|---------|---------|---------|---------|
| WEIGHT LBS | 99-135 | 136-176 | 177-214 | 215-251 | 252-300 |
| 19-20      | 1      | 2       | N/A     | N/A     | N/A     |
| 21-22      | 1      | 2       | N/A     | N/A     | N/A     |
| 23-24      | 1      | 2       | 3       | 4       | N/A     |
| 25-26      | 1      | 2       | 3       | 4       | 5       |
| 27-28      | 1      | 2       | 3       | 4       | 5       |
| 29-30      | N/A    | 2       | 3       | 4       | 5       |

### **HEEL ASSEMBLY**

If it is necessary to replace or service the heel component use Loctite 410 and torque to following specifications:

| Size  | Bolt Size | Heel Bolt Torque |
|-------|-----------|------------------|
| 19-21 | M6        | 14 Nm (10ft-lbs) |
| 22-30 | 1/4"      | 14 Nm (10ft-lbs) |

## COMPONENT OPTIONS

### 30 mm fiberglass tube kits

| Product No. | Cat. | Length      | Weight         |
|-------------|------|-------------|----------------|
| SFX00001    | 1–3  | 279 mm /11" | 292 g /0.3 oz  |
| SFX00002    | 1–3  | 381 mm /15" | 333 g /11.7 oz |
| SFX00003    | 4–5  | 279 mm /11" | 331 g /11.7 oz |
| SFX00004    | 4–5  | 381 mm /15" | 292 g /10.3 oz |

### Male pyramid adapter (Metric) w/bolts

| Product No. | Cat. | Weight        |
|-------------|------|---------------|
| SFX00165    | 1–5  | 110 g /5.5 oz |

### Shoe horn

| Product No.                                                                   |
|-------------------------------------------------------------------------------|
| CM100068                                                                      |
| <b>NOTE:</b> For assembly instructions please refer to the applicable IFU 's. |

## WARRANTY

Össur provides a warranty of 36 months for the Flex-Foot Assure foot module and 2 months for the foot cover kit.

## WATERPROOF IF USED WITH CARBON TUBE

A waterproof device allows use in a wet and/or humid environment and temporary (<30 mins.) submersion of up to 1m in fresh water. Exposure to fresh water under these circumstances shall have no harmful effect. Dry thoroughly after contact with fresh water. Fresh water: Includes tap water. Excludes salt and chlorinated water.

The components used with the foot need to be waterproof as well.

## WEATHERPROOF IF USED WITH PYRAMID

A weatherproof device allows use in a wet and/or humid environment but does not allow submersion. Fresh water splashing against the enclosure from any direction shall have no harmful effect. Dry thoroughly after contact with fresh water. Fresh water: Includes tap water. Excludes salt and chlorinated water.

## LIABILITY

The manufacturer recommends using the device only under the specified conditions and for the intended purposes. The device must be maintained according to the instructions for use. The manufacturer is not liable for damage caused by component combinations that were not authorized by the manufacturer.

## COMPLIANCE

This component has been tested according to ISO 10328 standard to two million load cycles. Depending on the amputee's activity this corresponds to a duration of use of two to three years. We recommend carrying out regular yearly safety checks

ISO 10328 - “P” - “m”kg ☆)

☆) Body mass limit not to be exceeded!

For specific conditions and limitations of use see manufacturer's written instructions on intended use!

In the standard mentioned, test levels (P) are assigned to a certain maximal body masses (m in kg). In some cases, which are marked with, no test level is assigned to the product related maximal body mass.

In these cases, the test loads have been adapted adequately on the basis of the specified load level.

| Category Össur |             |            |             |    |     |    |
|----------------|-------------|------------|-------------|----|-----|----|
| Category       | Weight (Kg) | Lable text |             |    |     |    |
| 1              | 61          | P3         | ISO 10328 - | P3 | 61  | kg |
| 2              | 80          | P3         | ISO 10328 - | P3 | 80  | kg |
| 3              | 97          | P4         | ISO 10328 - | P4 | 97  | kg |
| 4              | 114         | P4         | ISO 10328 - | P4 | 114 | kg |
| 5              | 136         | P4         | ISO 10328 - | P5 | 136 | kg |

- This manual is intended for use by a certified prosthetist.

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**EN – Caution:** Össur products and components are designed and tested according to the applicable official standards or an in-house defined standard when no official standard applies. Compatibility and compliance with these standard is achieved only when Össur products are used with other recommended Össur components. If un-usual movement or product wear is detected in a structural part of a device at any time, the patient should be instructed to immediately discontinue use of the device and consult his/her clinical specialist. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients. If any problems occur with the use of this product, immediately contact your medical professional.

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