



Instructions for Use




---

PRO-FLEX LP ALIGN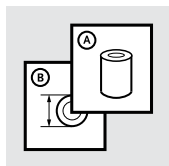

|    |                             |     |
|----|-----------------------------|-----|
|    |                             | 3   |
| EN | Instructions for Use        | 5   |
| DE | Gebrauchsanweisung          | 10  |
| FR | Notice d'utilisation        | 16  |
| ES | Instrucciones para el uso   | 22  |
| IT | Istruzioni per l'uso        | 28  |
| NO | Bruksanvisning              | 34  |
| DA | Brugsanvisning              | 39  |
| SV | Bruksanvisning              | 44  |
| EL | Οδηγίες Χρήσης              | 49  |
| FI | Käyttöohjeet                | 55  |
| NL | Gebruiksaanwijzing          | 61  |
| PT | Instruções de Utilização    | 67  |
| PL | Instrukcja użytkowania      | 73  |
| CS | Návod k použití             | 79  |
| TR | Kullanım Talimatları        | 84  |
| RU | Инструкция по использованию | 90  |
| JA | 取扱説明書                       | 96  |
| ZH | 中文说明书                       | 101 |
| KO | 사용 설명서                      | 106 |
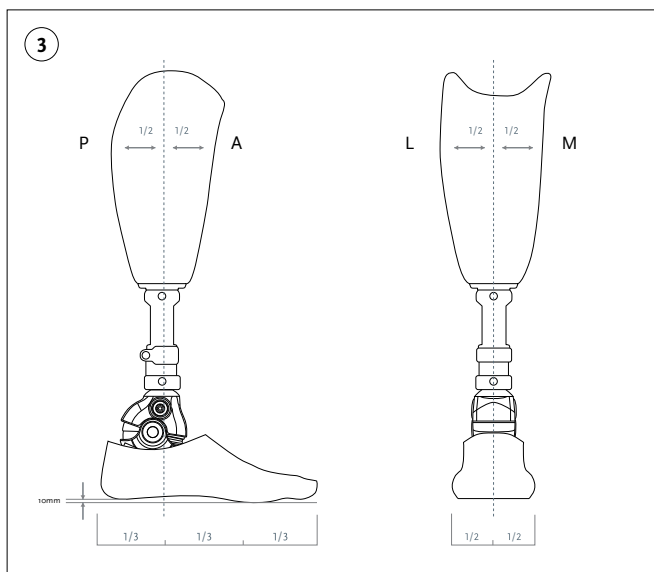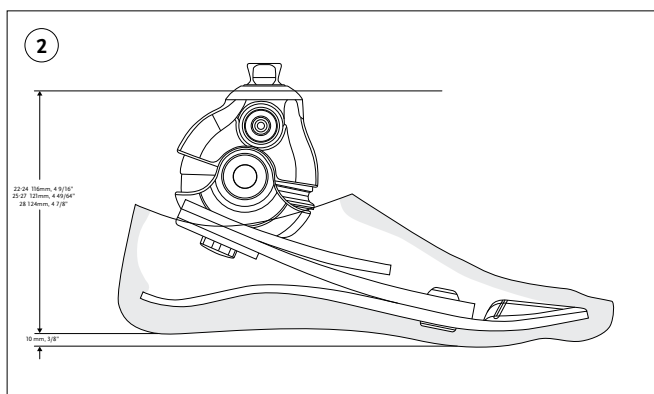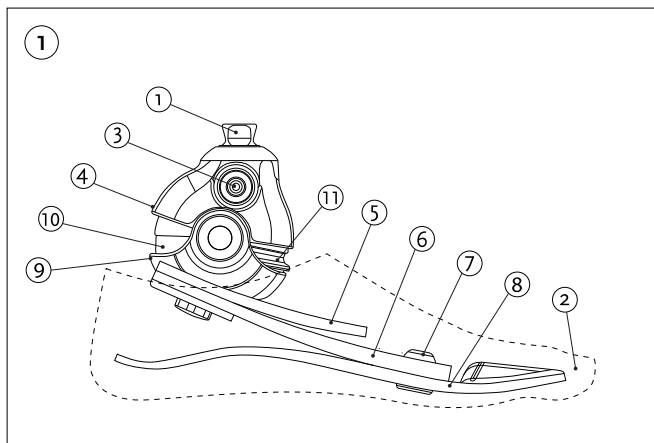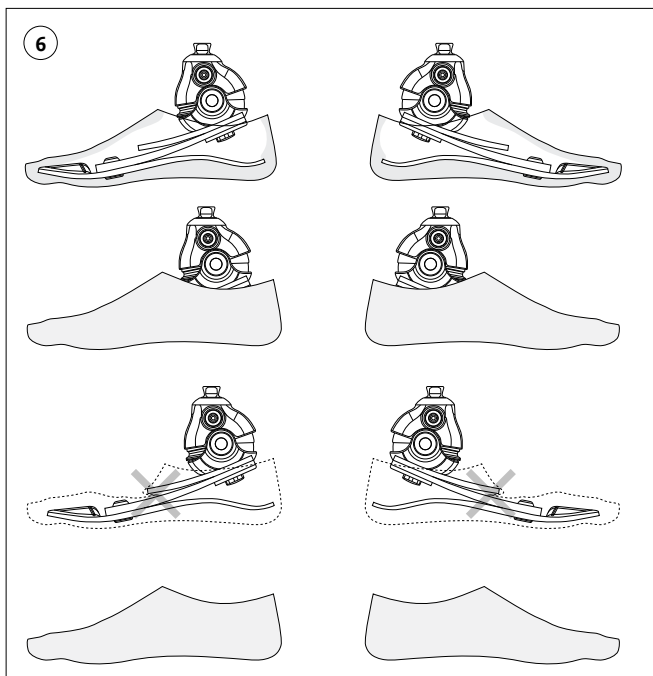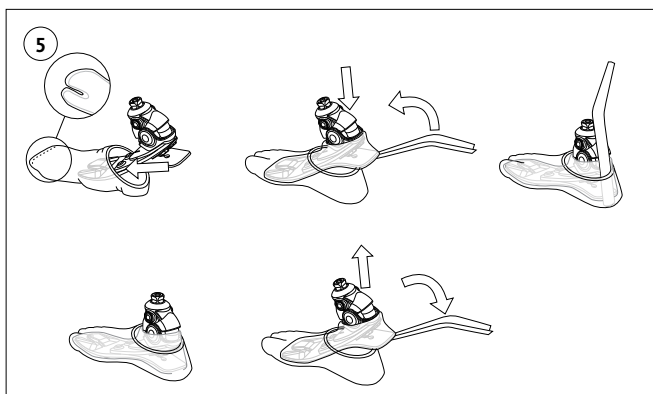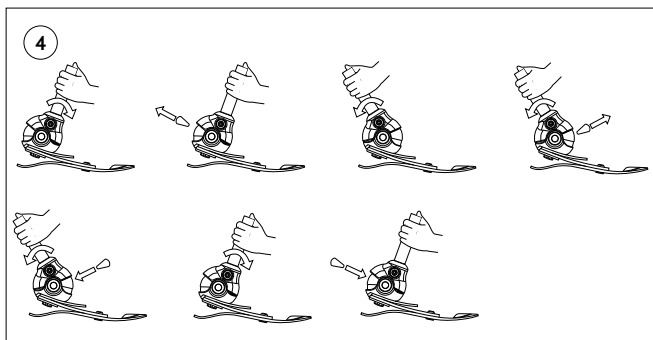
# ENGLISH

---

**Pro-Flex LP Align** is referred to as **the device** in the following document. This document provides important information on the indications for use, and handling of the device. This document is intended for a certified prosthetist and the user of the device. The device may only be fitted by a qualified practitioner authorized by Össur after completing the corresponding training. The practitioner must ensure that the user is able to operate the device.

## PRODUCT DESCRIPTION

The device is a prosthetic foot with user adjustable heel height. It is composed of an integrated hydraulic ankle unit, with a male pyramid and three composite blades. It features a sandal toe and a full-length composite blade. The bottom blade is secured with heel bolts.

The device consists of the following components: **(Figure 1)**

1. Male Pyramid
2. Foot Cover
3. Lock and release button
4. Upper housings
5. Top Blade
6. Middle Blade
7. Bottom Blade bolts
8. Bottom Blade
9. Lower Housings
10. Posterior Spacer
11. Anterior Spacer Wedge

## INDICATIONS FOR USE

Lower extremity amputations.

## CONTRAINDICATIONS FOR USE

None known.

## INTENDED USE

The device is a non-invasive, reusable prosthetic device, designed for single patient use.

The device is to be used exclusively for the exo-prosthetic fitting of amputations of the lower limb.

The device supports low to moderate impact levels.

## TECHNICAL SPECIFICATIONS (Figure 2)

The following specifications are for a size 24 category 5 foot with foot cover:

- Product weight with footcover : 703 g / 1.55 lbs
- Foot cover weight: 100 g / 0.22 lbs
- Build height: 116 mm / 4 9/16"
- Maximum user weight: 116 kg / 256 lbs

## CATEGORY SELECTION CHART

Please refer to the selection chart below to determine the appropriate category required according to Össur recommendations.

**NB:** Incorrect category selection may result in poor device function. Maximum user weight of 116 kg for low impact levels and 100 kg for moderate impact levels.

| Weight kg             | 45-52  | 53-59   | 60-68   | 69-77   | 78-88   | 89-100  | 101-116 |
|-----------------------|--------|---------|---------|---------|---------|---------|---------|
| Weight lbs            | 99-115 | 116-130 | 131-150 | 151-170 | 171-194 | 195-220 | 221-256 |
| Low Impact Level      |        |         |         |         |         |         |         |
| Sizes 22-24           | 1      | 1       | 2       | 3       | 4       | 5       | N/A     |
| Sizes 25-27           | 1      | 1       | 2       | 3       | 4       | 5       | 6       |
| Sizes 28              | 3      | 3       | 3       | 3       | 4       | 5       | 6       |
| Moderate Impact Level |        |         |         |         |         |         |         |
| Sizes 22-24           | 1      | 2       | 3       | 4       | 5       | N/A     | N/A     |
| Sizes 25-27           | 1      | 2       | 3       | 4       | 5       | 6       | N/A     |
| Sizes 28              | 3      | 3       | 3       | 4       | 5       | 6       | N/A     |

Foot sizes: 22-28

## SAFETY

### Warnings

Össur products and components are designed and tested according to applicable official standards, or an in-house defined standard where no official standard applies. Compatibility and compliance with these standards is achieved only when Össur products are used with recommended Össur components.

The device should be used with Össur adapters. The durability of the device cannot be guaranteed when non-Össur adapters are used.

In cases of damage, malfunction, changes in alignment, or functional loss, the user should immediately cease using the device, and return it to their practitioner. Noticeable functional loss may be the result of reduced stiffness or loss of support.

Improper handling (including repetitive high loads) can cause the device malfunction.

The device is weatherproof, therefore cannot be submerged in water (please refer to the *Weatherproof* section below).

### Precautions

The device must **NEVER** be used when lock/release button is unlocked.

Use of a shoe horn is recommended when donning or doffing the foot cover, to prevent damage to both the device and foot cover.

The user must keep fingers clear of the foot cover and blades when in use, as a pinch hazard exists.

### Weatherproof

A weatherproof device allows use in a wet and/or humid environment, but does not allow submersion. Fresh water splashing against the enclosure from any direction shall have no harmful effect. Dry thoroughly after contact with fresh water. Fresh water: Includes tap water. Excludes salt and chlorinated water.

For a weatherproof system, weatherproof components must be selected for use with the device.

## HEEL HEIGHT ADJUSTMENT

### Lock/Release Mechanism

- Press the medial (inner) side of the Lock/Release button\* to unlock the ankle.\*
- Adjust the ankle to the appropriate heel height by pressing on the toe or heel (it is also adjustable by hand when the prosthesis is not fitted, and resting on a table).
- The pylon or shin section should be perpendicular to the floor when the foot is in the shoe.
- When appropriate heel height adjustment is achieved, press the lateral (outer) side of the Lock/Release button to lock the ankle.
- Ensure the Lock/Release button is locked prior to weight bearing.  
**N.B.** The Lock/Release button may be stiff at first, or when not used for long periods of time. Pushing in and out several times will loosen the button. Similarly, the ankle may also be stiff, and should be moved up and down several times to loosen.

\***N.B.** The device **CANNOT** be used in the unlocked position for weight-bearing.

**N.B.** An extension for the Lock/Release button should not be used, as it may result in accidental release.

***Users are asked to skip to the section entitled FOOT COVERS AND SPECTRA SOCKS.***

***The following instructions are for the practitioner ONLY. The user is advised NOT to attempt these adjustments.***

### BENCH ALIGNMENT (Figure 3)

- Fit foot with selected cover and shoe.
- Ensure appropriate angles (flexion/extension and abduction/adduction).
- Divide the foot cover into 3 equal portions.
- The load line should fall at the junction of the posterior and the middle third portions as shown in figure 3.
- The 1/3 position is marked on the inside wall of the foot cover.

### DYNAMIC ALIGNMENT

The heel stores energy after initial contact, slowly releasing it at mid-stance. The forward momentum generated by this action results in the toe being loaded for optimum energy release at terminal stance.

The heel-to-toe action can be influenced by:

- Anterior/posterior positioning of the foot.
- Dorsi-plantarflexion.

### HEEL STIFFNESS

#### Heel too soft

##### *Symptoms*

- Foot comes to a flat position too early (user feels they are sinking into a hole).
- Extra energy required to climb up over the toe.
- Toe feels too stiff.
- Knee may hyperextend.

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## Heel too hard

### Symptoms

- Rapid heel-to-toe movement.
- At initial contact the user has poor control of prosthesis.
- User feels minimal energy return.
- Knee may become unstable.

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The posterior and anterior spacers may be replaced if they become worn. Please refer to the Össur Prosthetic Solutions catalog for replacement part details.

To replace:

- Remove the foot cover and Spectra sock from the device.
- Compress the posterior spacer to expose the anterior spacer **(Figure 4.1)\***.
- Remove the anterior spacer.
- Position the foot in maximum dorsiflexion to expose and remove the posterior spacer **(Figure 4.2)**.
- Replace with new posterior spacer, ensuring the spacer is fully seated **(Figure 4.3)**.
- Compress the posterior spacer to expose the anterior spacer **(Figure 4.4)\***.
- Replace with new anterior spacer, ensuring the spacer is fully seated.

\*Place the wedge included in the replacement kit between the bottom and middle blades to ease posterior spacer compression.

**N.B.** Ensure the wedge is removed before using the device.

## FOOT COVERS AND SPECTRA SOCKS (Figure 5)

- The device is designed to be used with a recommended foot cover and Spectra sock.
- Use a shoehorn when fitting and removing foot covers to avoid damage.
- Ensure the device is placed completely into the foot cover when fitting.
- The middle section of the spectra sock must cover the top blade of the device to ensure it doesn't interfere with the movement of the blades **(Figure 6)**.
- The foot cover should only be used with intended Össur products.

**N.B.** The device should not be used without a recommended foot cover and Spectra sock.

## MAINTAINANCE & CLEANING

- The device is weatherproof, but not resistant to corrosion. The device should not come into contact with salt water or chlorinated water. In cases where it does, the device must be rinsed with fresh water immediately, and dried with a lint-free cloth.
- The device is not designed to be used in extreme conditions such as diving or jumping into water.
- If device components get wet, parts should be dried with a lint-free cloth.

## DEBRIS BETWEEN BLADES

- Noise may occur if sand or debris are present between blades.

- The prosthetist should remove the Spectra sock and foot cover from the device, and use compressed air to remove sand or debris.

**N.B.** The device should always be used with a foot cover and Spectra sock to minimize dust and dirt from entering between carbon blades.

**USE ENVIRONMENT**

Environmental operating temperature: -15 °C (+5 °F) to 50 °C (122 °F).  
Operating humidity: 10-95% relative humidity.  
Shipping and storage temperature: -20 °C (-4 °F) to 70 °C (158 °F).

**LIABILITY**

The manufacturer recommends using the device only under the conditions specified and for the intended purposes. The device must be maintained according to the instructions for use. The manufacturer is not liable for damage caused by component combinations that were not authorized by the manufacturer.

**COMPLIANCE**

This device has been tested according to ISO 10328 standard to two million load cycles. Depending on the user’s activity this corresponds to a duration of use of two to three years. We recommend carrying out yearly safety checks. This device has been tested for moderate impact

ISO 10328 - “P” - “m”kg \*)

\*) Body mass limit not to be exceeded!

For specific conditions and limitations of use see manufacturer’s written instructions on intended use!

level use. Refer to category selection chart for low impact level use. In the standard mentioned, test levels (P) are assigned to certain maximal body masses (m). In some cases, which are marked with\*), no test level is assigned to the device related to maximal body

mass. In these cases, the test loads have been adapted adequately on the basis of the specified load level.

| ISO 10328 - label |             |                     |
|-------------------|-------------|---------------------|
| Category          | Weight (Kg) | Label text          |
| 1                 | 52          | ISO 10328–P3–52–kg  |
| 2                 | 59          | ISO 10328–P3–59–kg  |
| 3                 | 68          | ISO 10328–P3–68–kg  |
| 4                 | 77          | ISO 10328–P4–77–kg  |
| 5                 | 88          | ISO 10328–P4–88–kg  |
| 6                 | 100         | ISO 10328–P5–100–kg |

**WARRANTY**

Össur provides a warranty of 24 months for the device, and 6 months for the foot cover.

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**EN – Caution:** Össur products and components are designed and tested according to the applicable official standards or an in-house defined standard when no official standard applies. Compatibility and compliance with these standard is achieved only when Össur products are used with other recommended Össur components. If un-usual movement or product wear is detected in a structural part of a device at any time, the patient should be instructed to immediately discontinue use of the device and consult his/her clinical specialist. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients. If any problems occur with the use of this product, immediately contact your medical professional.

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