
Instructions for Use

PRO-FLEX® LP




Product number:



PLP0xyz

PLPUxyz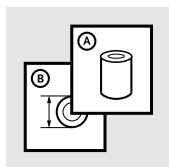

		3
EN	Instructions for Use	5
DE	Gebrauchsanweisung	9
FR	Notice d'utilisation	14
ES	Instrucciones para el uso	19
IT	Istruzioni per l'uso	24
NO	Bruksanvisning	29
DA	Brugsanvisning	33
SV	Bruksanvisning	38
EL	Οδηγίες χρήσης	42
FI	Käyttöohjeet	47
NL	Gebruiksaanwijzing	51
PT	Instruções de Utilização	56
PL	Instrukcja użytkowania	61
CS	Návod k použití	66
TR	Kullanım Talimatları	71
RU	Инструкция по использованию	76
JA	取扱説明書	81
ZH	中文说明书	86
KO	사용 설명서	90

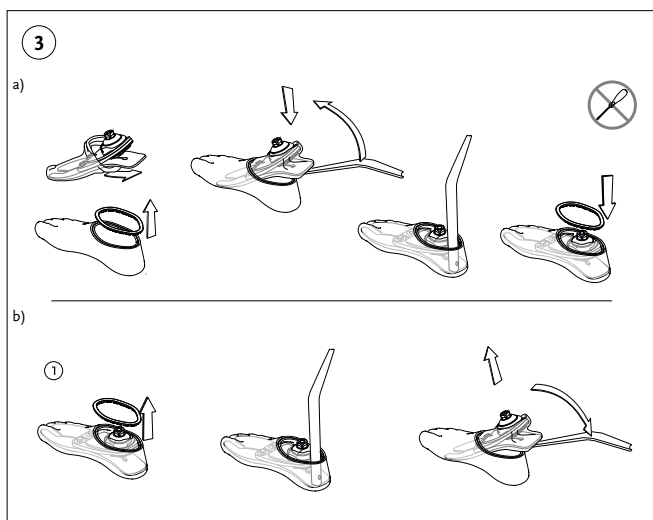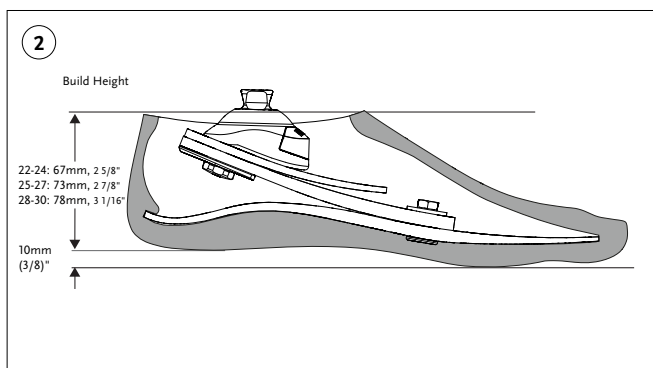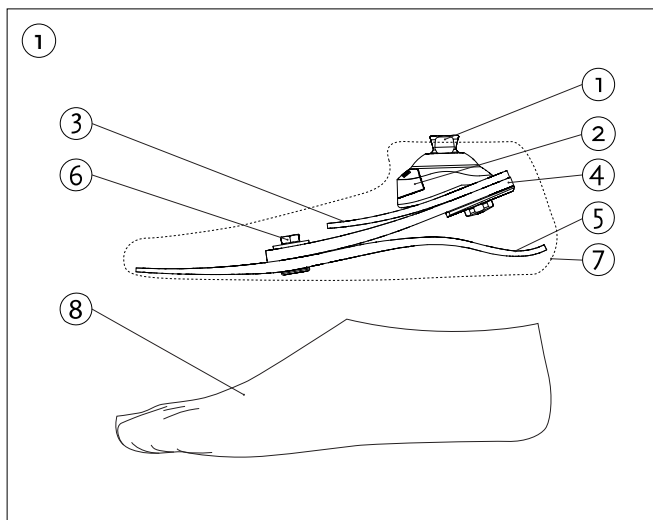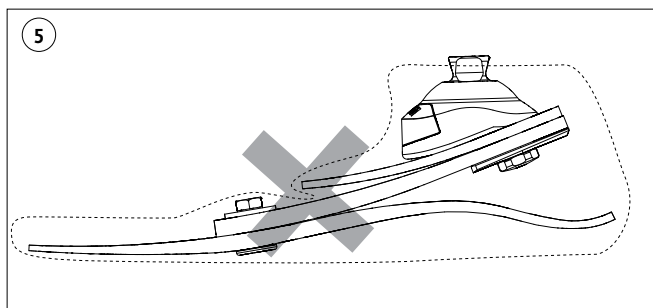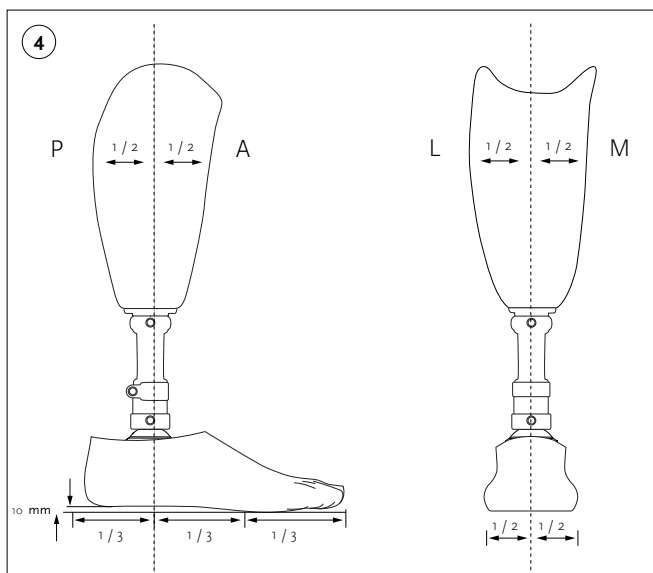ENGLISH

Pro-Flex LP is referred to as **the device** in the following document. This document provides important information on the indications for use, and handling of the device. It is intended for use by a certified prosthetist and the user of the device.

PRODUCT DESCRIPTION

The device is a prosthetic foot device with integrated male pyramid and three composite blades. It features a sandal toe and a full length composite blade. The bottom blade is secured together with heel bolts.

The device consists of the following components (**Figure 1**):

1. Male Pyramid Adapter
2. Pyramid cover
3. Top blade
4. Middle blade
5. Bottom blade
6. Bottom blade bolts
7. Spectra Sock
8. Footcover

INDICATIONS FOR USE

Lower limb amputation.

CONTRAINDICATIONS

None known.

INTENDED USE

The device is a non-invasive, reusable prosthetic device, designed for single patient use.

The device is to be used exclusively for the exo-prosthetic fitting of amputations of the lower limb.

The device supports low to high impact levels.

TECHNICAL SPECIFICATIONS

The following specifications are for a size 27 category 5 device with foot cover:

- Device weight with footcover : 647 g / 1.42lbs
- Foot cover weight: 165 g / 0.36 lbs
- Build height: 73 mm / 2 7/8"
- Maximum user weight: 166 kg / 365lbs

CATEGORY SELECTION CHART

Please refer to the selection charts below to determine the appropriate stiffness required according to Össur recommendations.

N.B.: Incorrect category selection may result in poor device function. Maximum user weight of 166 kg for low impact levels, 147 kg for moderate impact levels and 130 kg for high impact levels.

Weight kg	45-52	53-59	60-68	69-77	78-88	89-100	101-116	117-130	131-147	148-166
Weight lbs	99-115	116-130	131-150	151-170	171-194	195-220	221-256	257-287	288-324	325-365
Low Impact - Level										
Sizes 22-24	1	1	2	3	4	5	N/A	N/A	N/A	N/A
Sizes 25-27	1	1	2	3	4	5	6	7	8	9
Sizes 28-30	3	3	3	3	4	5	6	7	8	9
Moderate Impact - Level										
Sizes 22-24	1	2	3	4	5	N/A	N/A	N/A	N/A	N/A
Sizes 25-27	1	2	3	4	5	6	7	8	9	N/A
Sizes 28-30	3	3	3	4	5	6	7	8	9	N/A
High Impact - Level										
Sizes 22-24	2	3	4	5	N/A	N/A	N/A	N/A	N/A	N/A
Sizes 25-27	2	3	4	5	6	7	8	9	N/A	N/A
Sizes 28-30	3	3	4	5	6	7	8	9	N/A	N/A

SAFETY

Warnings

Össur products and components are designed and tested according to applicable official standards, or an in-house defined standard where no official standard applies. Compatibility and compliance with these standards is achieved only when Össur products are used with recommended Össur components.

In cases of damage, malfunction, changes in alignment, or functional loss, the user should immediately cease using the device, and return it to their practitioner. Noticeable functional loss may be the result of reduced stiffness or loss of support.

The device is not designed to be used under extreme conditions like diving or jumping into water.

The device is weatherproof, therefore cannot be submerged in water (please refer to the *Weatherproof* section below).

Weatherproof

A weatherproof device allows use in a wet and/or humid environment but does not allow submersion. Fresh water splashing against the enclosure from any direction shall have no harmful effect. Dry thoroughly after contact with fresh water. Fresh water: Includes tap water. Excludes salt and chlorinated water.

For a weatherproof system, weatherproof components must be selected for use with the device.

N.B.: The device is designed to be used with a Spectra sock, and recommended foot cover.

BENCH ALIGNMENT (Figure 4)

- Fit foot with selected cover.
- Adjust to appropriate heel height (using the shoe).
- Introduce appropriate socket angles flexion/ extension and abduction/adduction.
- Divide the foot cover into 3 equal portions.
- The load line should fall at the junction of the posterior and the middle third portions as shown in figure 3.
- The 1/3 position is marked on the inside wall of the foot cover.

DYNAMIC ALIGNMENT

The heel stores energy after initial contact, slowly releasing it at midstance. The forward momentum generated by this action results in the toe being loaded for optimum energy release at terminal stance. The heel to toe action can be influenced by:

- Anterior-Posterior positioning of device.
- Dorsi-Plantarflexion.
- Heel Stiffness.

HEEL STIFFNESS

Heel too soft

Symptoms

- Device will come to flat position too early (user feels he/she is sinking into a hole).
- Extra energy is required to climb up over the toe.
- Toe will feel too stiff
- Knee may hyperextend.

Action

- Shift socket anterior (or device posterior).

Heel too hard

Symptoms

- Rapid heel-to-toe movement.
- At initial contact the user has poor control of the prosthesis.
- Minimal energy return feeling.
- Knee may become unstable.

Action

- Shift socket posterior (or device anterior).

PROTECTIVE FILM ON PYRAMID

The protective film protects the pyramid from scratching. Remove the film when the fitting is complete.

FOOT COVERS AND SPECTRA SOCKS (Figure 3)

- The device is designed to be used with a recommended foot cover and Spectra sock.
- Use a shoehorn when fitting and removing foot covers to avoid damage (**Figure 3a and 3b**).
- Ensure the device is placed completely into the foot cover when fitting.
- The middle section of the Spectra sock must cover the top blade of the device to ensure it doesn't interfere with the movement of the blades (**Figure 5**).

MAINTAINANCE & CLEANING

N.B.: The device should always be used with spectra socks and foot covers to minimize dust and dirt from entering between carbon blades. It is important to fix the spectra sock around the pyramid location in order to prevent noise coming from sand entering between carbon blades.

- The device is weatherproof, but not resistant to corrosion. The device should not come into contact with salt water or chlorinated water. In cases where it does, the device must be rinsed with fresh water immediately, and dried with a lint-free cloth.
- If device components get wet, parts should be dried with a lint-free cloth.

DEBRIS BETWEEN BLADES

- Noise may occur if sand or debris are present between blades.
- The prosthetist should remove the *Spectra Sock* and foot cover from the device, and use compressed air to remove sand or debris.

N.B.: The device should always be used with a foot cover and *Spectra Sock* to minimize dust and dirt from entering between carbon blades.

USE ENVIRONMENT

- Environmental operating temperature: -15 °C (+5 °F) to 50 °C (122 °F).
- Operating humidity: 10-95% relative humidity
- Shipping and storage temperature: -20 °C (-4 °F) to 70 °C (158 °F).

LIABILITY

The manufacturer recommends using the device only under the specified conditions and for the intended purposes. The device must be maintained according to the instructions for use. The manufacturer is not liable for damage caused by component combinations that were not authorized by the manufacturer.

COMPLIANCE

This device has been tested according to ISO 10328 standard to two million load cycles. Depending on the user’s activity this corresponds to a duration of use of two to three years. We recommend carrying out yearly safety checks. This device has been tested for moderate impact level use. Refer to

ISO 10328 - “P” - “m”kg *)

*) Body mass limit not to be exceeded!

For specific conditions and limitations of use see manufacturer’s written instructions on intended use!

category selection chart for low and high impact level use. In the standard mentioned, test levels (P) are assigned to certain maximal body masses (m). In some cases, which are marked with*), no test level is assigned to the device related to maximal

body mass. In these cases, the test loads have been adapted adequately on the basis of the specified load level.

ISO 10328 - label		
Category	Weight (Kg)	Label text
1	52	ISO 10328–P3–52–kg
2	59	ISO 10328–P3–59–kg
3	68	ISO 10328–P3–68–kg
4	77	ISO 10328–P4–77–kg
5	88	ISO 10328–P4–88–kg
6	100	ISO 10328–P5–100–kg
7	116	ISO 10328–P6–116–kg
8	130	ISO 10328–P6–130–kg
9	147	ISO 10328–P6–147–kg

WARRANTY

Össur provides a warranty of 36 months for the device and 6 months for the foot cover.

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EN – Caution: Össur products and components are designed and tested according to the applicable official standards or an in-house defined standard when no official standard applies. Compatibility and compliance with these standard is achieved only when Össur products are used with other recommended Össur components. If un-usual movement or product wear is detected in a structural part of a device at any time, the patient should be instructed to immediately discontinue use of the device and consult his/her clinical specialist. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients. If any problems occur with the use of this product, immediately contact your medical professional.

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