



Instructions for Use

ICEROSS SEAL-IN[®] V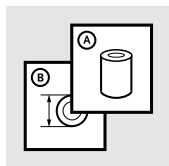

		3
EN	Instructions for Use	5
DE	Gebrauchsanweisung	10
FR	Notice d'utilisation	15
ES	Instrucciones para el uso	20
IT	Istruzioni per l'uso	25
DA	Brugsanvisning	30
SV	Bruksanvisning	35
EL	Οδηγίες Χρήσης	39
FI	Käyttöohjeet	44
NL	Gebruiksaanwijzing	49
PT	Instruções de Utilização	54
JA	取扱説明書	59
ZH	中文说明书	63
KO	사용 설명서	67

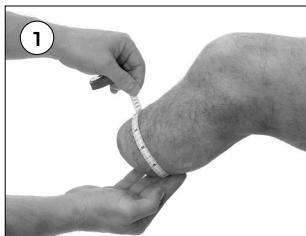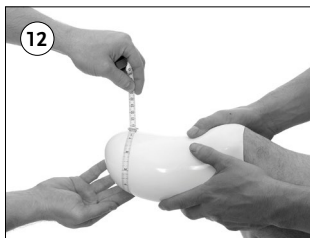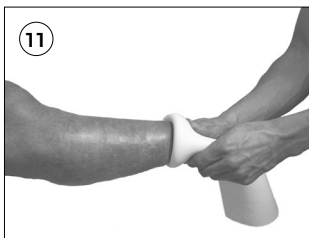ENGLISH

INTRODUCTION

Thank you for choosing a high quality silicone liner from Össur. We can assure you that you have made the right choice. Please carefully review and follow these instructions before fitting a patient with the liner or accessory products. Remember that informing the patient concerning proper handling and care of the liner, as set forth in these instructions, is important to achieve patient comfort and maximum product life.

INDICATION FOR USE

The Seal-In liner seal is designed to expand to accommodate daily volume fluctuations up to 4–6 ply while maintaining a secure suspension. If daily volume fluctuations are greater than 4–6 ply or the user experiences compromised suspension one or more Seal-In socks can be added to restore the socket fit (*See Section Iceross Seal-In Socks*).

CONTRAINDICATIONS FOR USE

- Extremely conical limbs (*see Section Iceross® Distal Cup*).
- When expecting drastic volume changes within short period of time, more than 8–10 ply.
- Very short residual limbs.

Minimum recommended length:

- To assess the minimum recommended length, measure the residual limb length from the mid patella tendon to the distal end, with knee in full extension.
- Seal-in V Standard: 9-12 cm depending on liner size
- Seal-In V High Profile: 13-15 cm depending on liner size

NOTE: Low posterior trim lines of the socket can affect suspension and should be assessed on individual basis.

LINER SIZING:

To determine the correct Iceross size (Figure 1):

1. Measure the circumference of the residual limb at 4cm from the distal end with the tissue hanging down.
2. Choose the Iceross size, which corresponds to the measurement or which, is immediately below the indicated measurement.
3. **Do Not** choose a liner size, which exceeds the measurement, or round up.

EXAMPLE: If the residual limb measures 31 cm at the indicated location, choose size 30 **NOT** size 32. Selecting the correct size is very important. If the liner is too tight, this may cause pistoning and/or numbness and blistering. A Seal-In liner that is too tight will also produce excessive ring tension to the residual limb from the seal. If the liner is too loose, increased perspiration and/or movement of the limb inside the liner may occur. Perspiration and movement can lead to blisters and rashes. If any of the above symptoms or any other indication of improper fit are noted, the user should be instructed to contact his/her physician or prosthetist immediately.

TRIMMING AN ICEROSS LINER

Iceross liners may be trimmed for user preference and to increase range

of motion (Figure 2). The Iceross CurveMaster is specifically designed for this function and to produce a smooth, rounded edge, which will minimize the risk of skin irritation and/or tears of the liner. Do not cut the liner below the level of the matrix or below the proximal trim-lines of the socket. Excessive trimming may reduce the natural suction between the liner and limb and/or compromise suspension.

DONNING THE ICEROSS SEAL-IN

Before donning a new liner clean it with lukewarm water, and dry it with lint free cloth as the natural oils in the silicone make it difficult to grip.

1. Invert the seal in the distal direction.
2. Turn the liner inside out by gripping the distal end from inside as shown in the illustration (Figure 3).
3. Grip the top of the liner and slide it over the hand until the inner surface is fully exposed (Figure 3).
4. After exposing as much of the distal end of the liner as possible, position it against the residual limb and with light compression roll upward onto the limb. Check that no air pockets are present (Figure 4).
5. Roll the liner all the way up the limb, taking care not to damage it with fingernails. Do not tug or pull (**Figure 6**).
6. Ensure the seal sits horizontally around their full circumference (Figure 5).

NOTE: Be very careful that the inside of the liner is clean, dry and free from any foreign objects that can cause skin irritation.

NOTE: The Silicone will soften with use, which will facilitate application.

DONNING THE PROSTHESIS

Iceross Seal-In is to be used with Icelock Expulsion Valve 551 (*See Section Icelock® Expulsions Valves*). The valve is specifically designed to maintain suspension when using Seal-In liners. A knee sleeve is not required.

1. Lightly spray the Seal around it's full circumference, and inside the socket for smoother entry (Figure 6).
2. Align the prosthesis as instructed by the prosthetist and push into the socket expelling air (Figure 7).

NOTE: If the user is having difficulty donning the prosthesis, more lubricant may be needed or the valve is blocked.

NOTE: If the user experiences the skin of the residual limb being stretched more to one side while donning the prosthesis, the distribution of the lubricant is inadequate around the full circumference of the Seal.

ICEROSS SEAL-IN SOCKS

If the user experiences insecurity due to compromised suspension due to volume fluctuations one or more Seal-In socks can be added to restore the socket fit.

Seal-in sock application:

1. Invert the seal completely in the distal direction (Figure 8).
2. Pull the Seal-in sock over the inverted seal until the distal end of the sock is level with the proximal border of the seal (Figure 9).
3. Reflect the seal over the end of the sock (Figure 10).

4. Don the prosthesis as normal.

NOTE: Loss of suspension may be encountered if a sock is placed over the seal.

PRODUCT NUMBER	DESCRIPTION
K-531xxx	Seal-In Socks

ICEROSS® CLEAN & SIMPLE LUBRICANT SPRAY

Iceross Clean & Simple lubricant spray has to be used to facilitate donning of the liner and the prosthesis. The lubricant is designed to evaporate quickly after donning to maintain maximum suspension. A spray pen accompanies the Seal-In Liner, small enough to fit in a pocket and/or a handbag. It can be refilled with the Iceross Clean & Simple lubricant spray.

WARNING: Do not apply to the skin. Avoid contact with eyes.

INGREDIENTS: Propanol and water.

PRODUCT NUMBER	DESCRIPTION
S-610250	Lubricant Spray

ICELOCK® EXPULSIONS VALVES

The Össur Icelock Expulsion Valves are designed and tested to continually expel air when loading the prosthesis, thus maintaining hypobaric suspension.

PRODUCT NUMBER	DESCRIPTION	BENEFITS
L-551002	Expulsion Valve 551	Small, cosmetic & easy installation

ICEROSS® DISTAL CUP

Iceross Distal Cups provide enhanced comfort for conical limbs with poor soft tissue coverage. They can also be used as a fitting tool in conjunction with the Iceross Original liners to ensure total contact between the silicone and residual limb. The DERMOGEL silicone in the Iceross Distal Cup conforms to the limb, thus minimizing pressure over bony prominence during gait.

To correctly fit a Distal Cup:

1. Measure the circumference of the residual limb at 4cm from the distal end (Figure 1).
2. Choose the Distal Cup two or three sizes smaller than the measurement.
3. Roll the Distal Cup directly over the limb (Figure 11).
4. When using an Iceross liner in combination with a Distal Cup, the liner measurement (4cm from the distal end) should be made with the Distal Cup in place (i.e. OVER the Distal Cup) (Figure 12).
5. Choose the correct Iceross liner size based on the measurement obtained with the Distal Cup in place

ICEROSS® PADS

Iceross Pads can be used to fill in any problem areas (i.e. voids or scars)

to ensure better fit and to minimize trapped air. Iceross Pads may also be used to distribute body pressure and to protect the skin. In some cases, custom-formed pads may be required for irregular shapes or problem areas. See Össur Prosthetic Product Catalogue for more information.

LINER CARE

Proper product care is important. The liner needs to be washed inside and out every day, following use. Your patient should be instructed to remove the liner, turn the liner inside out and wash it with pH balanced, 100% fragrance- and dye-free soap. The liner can also be machine washed (40°C/Hot) with a mild detergent. Fabric softeners or bleaches or other products or cleaning solutions may damage the liner and should **NOT** be used. After cleaning, the patient should rinse the liner thoroughly with water and wipe both sides dry with a lint-free cloth.

An Iceross liner can be used directly after washing. It does not need to dry over night. Always return the liner to its normal position with the seal facing out as soon as possible after cleaning. Ensure the seal is positioned correctly on the liner following washing. The seal must be clean and free from dirt prior to use.

NOTE: Misuse may result in potential loss of suspension. Iceross liners should always be checked for damage or wear. Any damage may weaken the effectiveness of the liner and should be reported to the user's prosthetist immediately.

Care must be taken by both the prosthetist and the patient not to expose a liner to glass or carbon fibers or other foreign particles. Such substances can become embedded in the silicone causing aggravation of the skin. Washing alone may not be sufficient to eliminate the problem and the patient should be instructed to return the liner to the prosthetist if the liner is inadvertently exposed to any foreign substances or chemicals.

DermoSil and DermoGel silicone contain Aloe Vera extract. This acts as a natural moisturizer and is known for its active healing and soothing effects on skin. Initially, the inner surface of the liner should be moist to the touch. For these reasons, the liner should not be left inside out for extended period or exposed to excessive heat or sunlight.

SKIN CARE

Daily cleaning of the residual limb is also essential. We recommend use of a mild liquid, pH balanced, 100% fragrance and dye-free soap. If dry skin is noted, apply a pH balanced, 100% fragrance- and dye-free lotion to nourish and soften the skin. Do **NOT** apply lotion to the residual limb immediately prior to donning the liner.

NOTE: Many common household or bath products, including soaps, deodorants, perfumes, aerosol or alcohol sprays or abrasive cleaners may cause or contribute to skin irritation and the patient should be so advised.

PRODUCT WARRANTY

Össur offers a written warranty on the Iceross Seal-In Liner for the indicated period from the invoice date. Said warranty shall be in lieu of all others, whether express or implied by law.

Iceross Seal-In Liner 6 months

For further information, please contact Össur or your local Össur distributor. You can also visit the Össur web page at **www.ossur.com**.

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EN – Caution: Össur products and components are designed and tested according to the applicable official standards or an in-house defined standard when no official standard applies. Compatibility and compliance with these standard is achieved only when Össur products are used with other recommended Össur components. If un-usual movement or product wear is detected in a structural part of a device at any time, the patient should be instructed to immediately discontinue use of the device and consult his/her clinical specialist. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients. If any problems occur with the use of this product, immediately contact your medical professional.

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